

PureWipe™

CPAP equipment cleaning wipes · 120 wipe tub



Read these instructions before first use and keep them with the product.



◆ Intended use

PureWipe is a pre-moistened wipe for cleaning the external surfaces of CPAP equipment: mask cushions and frames, the outside of air tubing, humidifier chambers and the outside of the device. It removes skin oils, sweat and everyday soil.


PLEASE NOTE

PureWipe is a **cleaning wipe, not a disinfectant**. It does not replace the periodic wash in warm soapy water set out by your mask and device manufacturers, and it cannot clean the inside of air tubing.

◆ Use on



Mask cushion

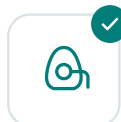

Frame and elbow



Tubing, outside



Chamber, outside



Device exterior


◆ Do not use on



Inside of tubing



Air outlet, blower, internal parts

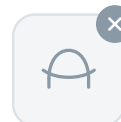

Headgear and fabric

◆ How to use



Open and draw

Flip open the cap and pull out one wipe. The next wipe waits in the slot.



Wipe the mask

Wipe both faces of the cushion seal, then the frame and elbow.



Wipe the rest

Outside of the tubing and chamber, and the device exterior. Never inside the air outlet.

4



Let it air dry

Leave the part to dry before use. No rinsing is needed.

5



Close the cap

Snap the cap shut so the remaining wipes stay moist.

6



Dispose

Put the used wipe in household waste. Do not flush.



Elarix Medical Ltd

Unit 3, Twenty4 Secure Storage, Armley, Leeds LS12 4HP, United Kingdom
Registered in England and Wales, company no. 16602969 ·
elarixmedical.co.uk



Warnings and precautions

- For external use on equipment only. Not for use on skin.
- Avoid contact with eyes.
- Keep out of reach of children.
- Do not put a wipe inside the air outlet, the blower or any internal part of the device.
- Follow the cleaning instructions issued by the mask and device manufacturer. Where they set out a method for a component, those instructions take precedence.
- Do not flush used wipes.

Suggested routine



Every morning

Mask cushion, both faces of the seal. Frame and elbow, or at least weekly.



Every week

Outside of the humidifier chamber and tubing. Device exterior and screen.

Keep the fuller wash in warm soapy water on its usual weekly cycle, and clean the inside of the tubing with a brush or a soak.

Storage and disposal

Storage	5 °C to 25 °C, cool and out of direct sunlight
After use	Close the cap to keep the wipes moist
Shelf life	Unopened TBC · after opening TBC
Disposal	Household waste. Do not flush

Product details

Contents	120 pre-moistened wipes
Ingredients	Water, proprietary cleaning solution, aloe vera extract, vitamin E. Unscented. No alcohol. No latex.
REF	ALV54001
Batch and use-by date	Printed on the tub TBC

Symbols



Manufacturer



Date of manufacture



Use-by date



Batch code



Catalogue number



Temperature limit, 5 °C to 25 °C



Keep away from sunlight



Consult instructions for use



Caution



Keep out of reach of children



Do not flush

PROBLEMS AND FEEDBACK

Report any problem with this product to Elarix Medical, quoting the batch code on the tub. Contact TBC



Elarix Medical Ltd

Unit 3, Twenty4 Secure Storage
Armley, Leeds LS12 4HP
United Kingdom



Elarix Medical Ltd

Unit 3, Twenty4 Secure Storage, Armley, Leeds LS12 4HP, United Kingdom
Registered in England and Wales, company no. 16602969 ·
elarixmedical.co.uk