

CPAP CARE · EQUIPMENT CLEANING

# PureWipe™

CPAP equipment cleaning wipes

120 pre-moistened wipes in a lidded dispensing tub

Alcohol-free

Latex-free

Unscented

No rinse



120

wipes per tub

≈ 4

months of nightly cushion  
cleaning

0

alcohol, latex or fragrance

UK

manufactured by Elarix  
Medical

## Overview

PureWipe is a pre-moistened wipe for the daily cleaning of CPAP equipment surfaces. It removes the film of skin oil and sweat that builds on a mask cushion overnight, which softens the seal and leads to leaks. The formula contains no alcohol and leaves no residue, so cleaned parts go straight back into use without rinsing.

## Key features



### Contains no alcohol

Repeated use does not dry, cloud or harden cushion silicone.



### Leaves no residue

Equipment goes straight back into use. No rinsing step.



### Unscented

No fragrance on a surface the user breathes through overnight.



### Contains no latex

Suitable for users with a latex sensitivity.



### Aloe vera and vitamin E

Chosen for surfaces that rest against facial skin.



### One tub, whole set

Cushion, frame, tubing exterior, chamber and device exterior.

## Ordering information

| DESCRIPTION                  | REF      | PRODUCT CODE | PACK      |
|------------------------------|----------|--------------|-----------|
| PureWipe CPAP cleaning wipes | ALV54001 | ALV-PW-120   | 120 / tub |



GTIN · EAN-13



Elarix Medical Ltd

Unit 3, Twenty4 Secure Storage, Armley, Leeds LS12 4HP, United Kingdom  
Registered in England and Wales, company no. 16602969 ·  
elarixmedical.co.uk

◆ Specifications

|                        |                                                                    |
|------------------------|--------------------------------------------------------------------|
| REF / SKU              | ALV54001                                                           |
| GTIN (EAN-13)          | 5065027735095                                                      |
| Pack size              | 120 wipes per tub                                                  |
| Format                 | Pre-moistened wipes, lidded dispensing tub                         |
| Wipe dimensions        | TBC                                                                |
| Wipe substrate         | TBC                                                                |
| Tub dimensions         | TBC                                                                |
| Net / packed weight    | TBC                                                                |
| Ingredients            | Water, proprietary cleaning solution, aloe vera extract, vitamin E |
| Fragrance              | Unscented                                                          |
| Alcohol                | None                                                               |
| Natural rubber latex   | None                                                               |
| Rinsing after use      | Not required                                                       |
| Container              | Polypropylene tub and lid                                          |
| Country of manufacture | United Kingdom                                                     |

◆ Storage and disposal

**Store between 5 °C and 25 °C**  
Keep cool.

**Keep out of direct sunlight**

**Shelf life, unopened** TBC  
**After opening** TBC

**Reseal the lid after each use**  
Keeps the remaining wipes moist.

**Do not flush**  
Dispose of used wipes in household waste.

◆ Logistics

|                  |     |
|------------------|-----|
| Units per case   | TBC |
| Case dimensions  | TBC |
| Case weight      | TBC |
| Cases per pallet | TBC |
| Commodity code   | TBC |

◆ Suitable surfaces and suggested routine

|                                                                                     | COMPONENT                   | WITH PUREWIPE                                 | WIPE |
|-------------------------------------------------------------------------------------|-----------------------------|-----------------------------------------------|------|
|  | Mask cushion                | Every morning, both faces of the seal         | ✓    |
|  | Mask frame and elbow        | Every morning, or at least weekly             | ✓    |
|  | Humidifier chamber, outside | Weekly, alongside the maker's washing routine | ✓    |
|  | Air tubing, outside         | Weekly. Inside: brush or soak                 | ✓    |
|  | Device exterior and screen  | Weekly, avoiding air inlet and outlet         | ✓    |
|  | Headgear and fabric parts   | Not wiped. Hand wash to mask instructions     | ✗    |

PRODUCT STATUS

A cleaning wipe, not a disinfectant. It removes soil and skin oils and makes no antimicrobial claim. Mask and device manufacturers' cleaning instructions take precedence.

Regulatory classification **PENDING**

DOCUMENTATION

Instructions for use ALV-IFU-54001

Full INCI declaration On request

Safety data sheet **TBC**

Specifications can change without notice. Ask Elarix Medical for the current technical file.