

OxiPro OX2

Fingertip Pulse Oximeter

Quick Start Guide V. 1.3

Product
Support




www.alcosense.co.uk/oxipro
info@alcosense.co.uk

UK: 01628 778885
IRL: 01 513 7065
Lines open Mon-Fri
0900-1700


1 INSTALL BATTERIES

Turn oximeter over, slide the battery cover as shown to remove it. Insert 2 x AAA batteries (included) and replace the battery cover.

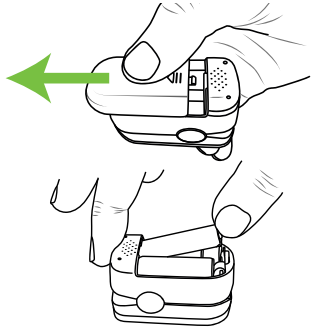


2 PERFORMING A TEST

- 1) Clean hands
- 2) Remove nail varnish or false nails, they blocks the signal.
- 3) Make sure your fingers and hands are warm to the touch.

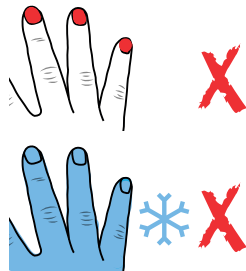


Take a reading when you are relaxed, sitting or laying down.
Rest your testing hand on a fixed surface or part of your body when testing.
Remain still while testing.

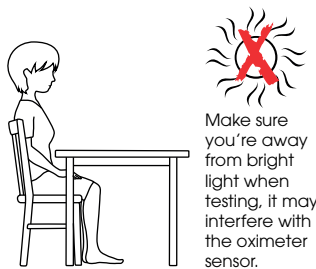


Open the oximeter by squeezing the end together.

Insert the **middle finger of your dominant hand** until you feel the rubber stop.

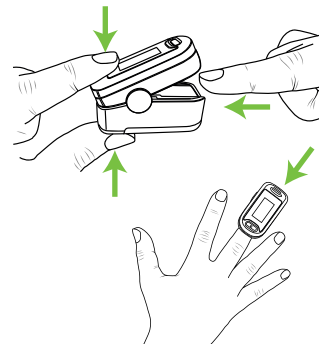

IMPORTANT

- 1) **SIT OR LIE STILL WHILE TESTING.**
- 2) **REST YOUR HAND ON A SURFACE WHILE TESTING.**
- 3) **TURN THE OXIMETER ON AFTER IT IS ON YOUR FINGER, BY PRESSING THE BUTTON ON THE FRONT.**
- 4) **WAIT FOR THE READING TO STABILISE (UP TO 60 SECONDS) TAKING THE HIGHEST READING.**

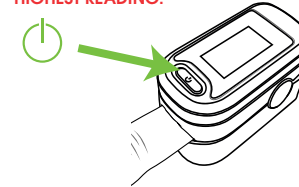

If you get a low or inconsistent blood oxygen reading

Blood flow quality can significantly vary between fingers. Typically, the best fingers are the middle fingers followed by the index finger, but this can vary from person to person.

If you get an unstable reading, low reading or weak signal, try another finger or hand as oximeters can be used on any finger. Wait up to 60 seconds until the blood oxygen reading stabilises (shows the same reading for 8-10 seconds) and take the highest reading shown.

3 UNDERSTANDING THE RESULTS

Wait for the reading to stabilise.

10 to 60 secs

Pulse graph
Heart rate (Beats per minute)



Blood oxygen reading (SpO₂)

Perfusion Index (See full manual)


NOTE: Screen layout may vary

Take the stable blood pressure reading from the oximeter and reference it against the chart below (Source: NHS).

If you have a reading of 94% or below (after the reading has stabilised), change fingers and take a second reading. Making sure you are still, with your hand supported on a stable surface, remove all clear or coloured nail varnish and wait up to 60 seconds for the reading to stabilise again.

95% to 100%	93%-94%	92% or less
This is the normal range for healthy adults without existing lung conditions	Seek urgent medical advice or call 111	Go to A&E immediately or call 999


TOP TIP! You can change the orientation of the screen through several options by clicking the power button while testing.

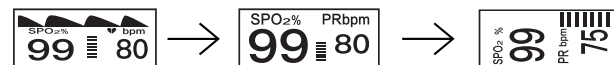

4 TURNING OFF

Remove your finger from the oximeter. It will turn off automatically in 10-20 seconds

10-20 secs

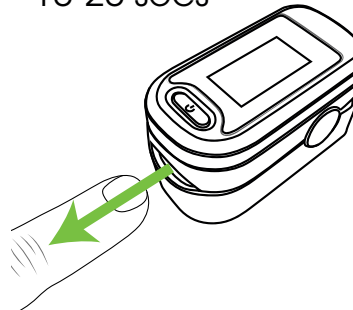

5 TROUBLE SHOOTING

Oximeter does not turn on	Check that the batteries have been inserted the right way round. If the batteries are correctly inserted try new batteries. Contact us.
My reading seems incorrect, unstable or inconsistent	See the section of this quick start guide with the explanation mark symbol at the top (far right on the first page). In addition to this, it could be as a result of movement whilst the reading is being taken or light leakage into the sensor from the side of the finger. It can take up to 1 minute for the reading to stabilise depending on your skin type and the ambient light conditions. Make sure your hand is supported on fixed surface while testing and stay still, if it still does not stabilise try a different finger and give the oximeter a slight squeeze when it is on your finger to increase the light seal around the sensor. Also check you are not wearing clear or coloured nail varnish or false nails and if so, remove it from the test finger. Test away from bright light sources. If this doesn't resolve it, please contact us.
I'm stuck in the settings menu	The settings menu is accessed by pressing and holding the power button. Just leave the oximeter for 10-20 seconds and it will power off. See the main manual to find out how to use the menu system.
I can't turn it off	The oximeter will turn off automatically in 10-20 seconds if a finger is not inserted. You cannot turn it off manually.
It turned itself off	The oximeter turns itself off automatically if it does not sense a signal in 10-20 seconds. Put the oximeter on your finger before turning on.
Anything else	Please contact us and we'll be happy to help.